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Kontakt podaci proizvođača / lokalni kontakt

Cardio Medical Zagreb d.o.o.
Utinjska 40, Zagreb
01/7898-287

Kontakt podaci HALMED-a

Molimo Vas da štetne događaje povezane s medicinskim proizvodima prijavite HALMED-u putem obrazaca dostupnih na Internetskim stranicama HALMED-a na adresi: <http://www.halmed.hr/O-HALMED-u/Obrasci/>.

Popunjeni obrazac možete dostaviti:
telefaksom na broj: + 385 1 4884 110
na adresu elektroničke pošte: medpro@halmed.hr

Appendix A: Recommendations for Patients for Prevention of Excessive Static Electricity

The following clarifications will be added to the Instructions for Use and Patient Handbook. Please contact all current HeartMate 3 patients and provide them with this additional information. Training should be provided to all new HeartMate 3 patients on this information until the updated IFU and Patient Handbook are available.

GENERAL WARNINGS

High levels of static electricity may damage or harm the system and cause your pump to stop. Use battery power before doing activities that can cause static electricity. High levels of static electricity can occur while:

- Folding or changing bedsheets
- Taking laundry out of a dryer
- Dragging your feet on a carpet
- Touching the screens of older TVs or computer (LCD and LED screens are less likely to cause static electricity)

Also, use a humidifier, dryer sheets, fabric softener, and skin moisturizer to reduce the buildup of static electricity.

Static Electricity

When you are not sleeping or resting, it is recommended to use battery power instead of the Mobile Power Unit™ to power your system. Using battery power can reduce the risk of system damage from high levels of static electricity.

Static electricity occurs when two objects come into contact. You can receive a static shock when performing the activities listed under GENERAL WARNINGS. Fabrics like wool, silk, and synthetic materials can buildup static electricity. Use cotton fabric where possible.

Static electricity is more common when the air is dry (relative humidity less than 20%). Cold weather and home heating systems can make air drier. A humidifier can make air less dry and reduce static electricity.

WHAT YOU SHOULD DO:

Using battery power will help avoid system damage from static electricity. If you're on the Mobile Power Unit, switch to battery power when doing things that cause static electricity.

Reduce static electricity, with products such as:

- A humidifier to add moisture to the air
- Dryer sheets and fabric softeners to reduce the buildup of static electricity in clothes and bedsheets
- Anti-static spray on carpets and other materials to reduce the buildup of static electricity
- Skin moisturizers to make your skin less likely to build up static electricity
- Fabrics with cotton, which cause less static electricity than wool, silk and synthetic materials. Choose cotton for clothing and bedsheets instead of wool, silk, and synthetic materials.