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Upute za upotrebu za LIJEČNIKA

www.heartware.com/clinicians/instructions-use

1.4 Warnings (continued)

⚠️ WARNINGS

- 9. **WARNING!** DO NOT plug the controller into an AC wall outlet during showers: to eliminate the possibility of a severe electrical shock, it should be connected to two batteries.
- 10. **WARNING!** DO NOT allow patients to take a bath or swim, as this may damage HeartWare™ HVAD™ System components and/or result in driveline exit site infection.
- 11. **WARNING!** DO NOT submerge HeartWare™ HVAD™ System components in water or other fluid as this may damage them. If this happens, contact HeartWare.
- 12. **WARNING!** DO NOT allow water or other fluids to enter the controller, power adapters, batteries, battery charger or connectors, as this may damage HeartWare™ HVAD™ System components. If this happens, contact HeartWare.
- 13. **WARNING!** DO NOT use any components other than those supplied by HeartWare with the HeartWare™ HVAD™ System, as this may affect HeartWare™ HVAD™ System operation.
- 14. **WARNING!** Damaged equipment should be reported to HeartWare and inspected.
- 15. **WARNING!** DO NOT rely only on flow estimation to assess cardiac output. An average estimated flow on the monitor or Controller Display of less than 2.0 L/min, or greater than 10.0 L/min may indicate an electrical fault, incorrect hematocrit entry or an occlusion and/or thrombus or other materials (e.g. tissue fragments) in the device. Inaccurate assessment of HVAD® Pump flow may lead to less than optimal treatment.
- 16. **WARNING!** DO NOT grasp the driveline cable as this may damage the driveline. To remove the driveline from the controller, first pull back the driveline cover then grasp and pull the driveline connector.
- 17. **WARNING!** DO NOT disconnect the driveline from the controller or the pump will stop. If this happens, reconnect the driveline to the controller as soon as possible to restart the

2.6 Carrying Cases and Shower Bag (continued)

HeartWare® Shower Bag

A shower bag is available for use in conjunction with the HeartWare™ HVAD™ System. To ensure safe and appropriate use of the shower bag, all patients and caregivers should be trained on shower bag operation prior to use.

- WARNING!** DO NOT allow patients to shower until they have received permission from their clinician to do so. Patients who shower must use the HeartWare® Shower Bag.
- WARNING!** DO NOT allow hearing impaired patients to shower unless their caregiver is close by to hear alarms.
- WARNING!** DO NOT plug the controller into an AC wall outlet during showers: to eliminate the possibility of a severe electrical shock, it should be connected to two batteries.
- WARNING!** DO NOT allow patients to take a bath or swim, as this may damage HeartWare™ HVAD™ System components and/or result in driveline exit site infection.
- WARNING!** DO NOT submerge HeartWare™ HVAD™ System components in water or other fluid as this may damage them. If this happens, contact HeartWare.
- WARNING!** DO NOT allow water or other fluids to enter the controller, power adapters, batteries, battery charger or connectors, as this may damage HeartWare™ HVAD™ System components. If this happens, contact HeartWare.
- WARNING!** DO NOT use any components other than those supplied by HeartWare with the HeartWare™ HVAD™ System, as this may affect HeartWare™ HVAD™ System operation.
- WARNING!** Damaged equipment should be reported to HeartWare and inspected.

Upute za upotrebu PRIRUČNIK ZA BOLESNIKA

6.0 Living with the HeartWare™ HVAD™ System

6.4 Washing and Showering

Your clinician will let you wash your incisions after your wounds have healed. When you wash, the controller, batteries and connectors must be protected from water and you should take care so that water doesn't run along the driveline into the controller. The exit site should also be kept as dry as possible. Keeping the exit site dry helps avoid infections.

Your clinician will decide if it is safe for you to shower. **If your clinician gives you permission to shower, you must use the HeartWare® Shower Bag to protect the controller and batteries.**

WARNING! DO NOT shower until your clinician tells you it is safe to do so. If you receive permission to shower, you must use the HeartWare® Shower Bag. If your hearing is impaired and/or you cannot hear the controller alarms without the use of a hearing aid, make sure your caregiver will be close by to hear alarms.

WARNING! DO NOT plug the controller into an AC wall outlet during showers; it should be connected to two batteries.

WARNING! DO NOT take a bath or swim.

WARNING! DO NOT submerge any HeartWare™ HVAD™ System component in water.

WARNING! DO NOT allow water or other fluids to enter the controller, power (AC/DC) adapters, batteries, battery charger, or connectors. If this happens, contact your clinician.

HeartWare® Shower Bag

The HeartWare® Shower Bag provides the ability to comfortably and securely shower with your HeartWare™ HVAD™ System.

The shower bag is water resistant, not water proof and protects the controller and batteries from direct water spray and moisture. The shower bag permits one (1) controller and two (2) batteries to be placed into a single compartment.

Figure 13

For additional information on using the shower bag, see Section 6.5.

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