

Carl Zeiss Meditec AG 07740 Jena

MED-Q

Carl Zeiss Meditec AG Goeschwitzer Str. 51-52 07745 Jena

Germany

Phone: +44 (0) 1506-401018

e-mail: Laura.Allan@zeiss.com

Your ref .:

Yours of:

Our ref.: R2019-002 Date: 02/04/2019

Laura Allan

ACTION REQUIRED: FIELD SAFETY NOTICE

Product Effected: 003500-0026-893 Visthesia 1.0% batch 140692AA

Dear Customer,

Division/Dept.:

Your contact:

You are using the ZEISS Ophthalmic Viscosurgical Devices (OVDs). As you know, high quality and innovation is our main goal, but the safety of your patients is our first priority. To ensure that you are able to continue reliably using our products, Carl Zeiss Meditec is issuing a Field Safety Notice regarding the product listed above.

With this letter, we would like to inform you that we have received one report from the field that an OVD has been found, whose 27G Cannula was not the correct shape. Our initial investigation has identified a supplier manufacturing error which has resulted in a small number of cannula not being correctly bent and we have identified excess Silicone Oil coating on the tip of the cannula. Silicone Oil is biocompatible and a thin coating layer is part of the specification. In order to eliminate potential risk to the patient, any such cannulas should not be used.

Actions to be taken:

Please check the cannulas of Visthesia 1.0% batch 140692AA before use and discard them, if they are not properly bent. Your Zeiss representative will contact you shortly to arrange a visit and replace affected OVDs.

Affected Products:

Product:

Visthesia 1.0%®

Lot Number:

140692AA

Expiry Date:

2021-02

Ref:

A090004

Chairman of the Supervisory Board:

Dr. Michael Kaschke

Address of Record: Goeschwitzer Str. 51-52 07745 Jena, Germany Tel.: +49 36 41 220-0

Deutsche Bank Jena Account: 624536900 (BIC 820 700 00) S.W.I.F.T.-Code: DEUT DE 8EXXX IBAN: DE90820700000624536900

Commercial Register: Local Court Jena HRB 205623 VAT-ID. No. DE 811 922 737 WEEE-Reg.-Nr. DE55298748

Board of Management: Dr. Ludwin Monz, President and CEO Ian Willem de Cler

Address for Delivery: Carl Zeiss Meditec AG Carl-Zeiss-Promenade 10 Justus Wehmer 07745 Jena, Germany

Commerzbank Jena Account: 258072800 (BIC 820 400 00) S.W.I.F.T.- Code: COBADEFEXXX IBAN: DE31820400000258072800



Order Code:


003500-0026-893

Incorect shape (Needle does not have a 35° angle 9mm from the tip):





Correct shape:

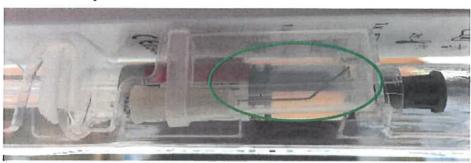



We understand that this notification is highly inconvenient for you. We would like to apologize and thank you in advance for your support with regards to this Field Safety Notification. Please be assured that we are investigating the non-conformity along with our supplier to determine root cause and will implement the necessary corrective actions to prevent this from happening again.

We thank you very much for your attention, your consequent verifications and your continuous support.

If you have any further questions please do not hesitate to contact us.

Best regards

Carl Zeiss Meditec AG

i.V.

Colin Blair

Quality Management Representative

i.V.

Christina Jakob

Director Product Management Cataract Implants &

Consumables