

# **URGENT: FIELD SAFETY NOTICE - PAS-18-1162**

BD Vacutainer® SST™ II *Advance* Plus Blood Collection Tubes Catalogue Number: 368879, Lot Number: 6319963

15<sup>th</sup> January 2018

Attention: Medical Device Safety Officer, Risk Manager, Laboratory Director, Head of

Supply Chain, Phlebotomy Manager, Clinical Studies Director, Researchers,

Clinical Pathologists, Nursing Manager.


This letter contains important information which requires your **immediate** attention.

## **Details of affected devices and description of the problem:**

BD Life Sciences – Preanalytical Systems is issuing a recall for catalogue code 368879, BD Vacutainer® SST™ II *Advance* Plus Blood Collection Tubes, lot number 6319963.

BD has received feedback from customers that the fill line is in the incorrect position (see below image) for tubes associated with this lot number. During investigation of these complaints, BD has become aware that the labelling for this lot of tubes has the incorrect shelf life. The shelf life is listed as 18 months, as opposed to the correct shelf life, 10 months.

Product associated with this lot expired in September 2017, but the label indicates the product has remaining shelf life, through May 2018. The use of an expired tube can result in lower draw volume due to the loss of vacuum in the tube. Furthermore, because the fill line is incorrectly located on the tube, a lower draw volume (e.g. under filled tube) would be difficult to detect prior to clinical testing. A lower draw volume could affect the blood to additive ratio, with the potential for erroneous results. Results should be correlated with the patient's clinical picture and repeat testing may be required.

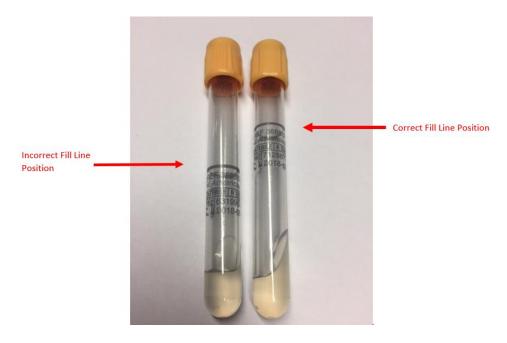



#### Table 1 – Affected product details

Catalogue Number	Product Description	Lot number	Incorrect expiry date on Labelling	Correct Expiry Date
368879	BD Vacutainer® SST™ II <i>Advance</i> Plus Blood Collection Tubes	6319963	2018-05	2017-09

This Field Safety Notice only affects the catalogue and lot number in Table 1.

To locate the catalogue and lot number, refer to Appendix 1.

### Advice on action to be taken:

- Stop use of the affected product and guarantine all stock as soon as possible.
- Inform appropriate personnel in your organisation to discontinue use of the affected lot.
- BD recommends discussing the content of this letter with your laboratory director regarding the need to review previous test results associated with this lot of product. Our records indicate your organisation received this product on <XXX> date.
- Complete and return the acknowledgement form (page 3) as soon as possible or <u>no</u> <u>later than the 1st February 2018</u> following the instructions, in exchange for replacement product or credit.
- If you destroy affected product, the certificate of destruction form (page 4), should also be sent.

## **Transmission of this Field Safety Notice**

Please maintain awareness of this notice in your organisation for an appropriate period to ensure effectiveness of the corrective action.

### **Contact Reference Person**

If you have any questions about the product please contact your local BD representative or the BD office on **tel no.** or **email**.

BD Life Sciences – Preanalytical Systems is committed to providing quality products to our customers and we have undertaken appropriate internal corrective actions. We apologise for the inconvenience this situation may cause.

We confirm that the appropriate regulatory agencies have been informed of these actions.

Yours sincerely,

Lorna Darrock EMEA Regulatory Affairs and Compliance Manager BD - Preanalytical Systems



# **Field Safety Notice Acknowledgement Form**

# BD Vacutainer® SST $^{\text{TM}}$ II Advance Plus Blood Collection Tubes Catalogue number 368879

	de Hamber 666676
Please read in conjunction with Field Safety by fax to: fax no. as soon as possible or no la	Notice PAS-18-1162 and return form to e-mail or ater than the 1st February 2018.
<ul> <li>product or credit (tick appropriate box be</li> <li>This notice has been read and understood (Fill out and return this form to BD at fax/e</li> </ul>	tory and will return in exchange for replacemen low). od and distributed to all appropriate personnel. or and above and return the product with a copy of this 8-1162, BD, DC3, Laagstraat 57, B- 9140 Temse
<ul> <li>in exchange for replacement product or of</li> <li>This notice has been read and understood</li> </ul>	y and I confirm that this has been destroyed on site credit (tick appropriate box below).  In and distributed to all appropriate personnel.  Certificate of Destruction form, and provide evidence.
<ul> <li>Option 3</li> <li>I have NO affected product left in inventor</li> <li>This notice has been read and understood (Fill out and return this form to BD at fax/e-inventor)</li> </ul>	od and distributed to all appropriate personnel.
Organisation / Hospital / Clinic :	
Department (if applicable):	
Address:	
Postcode:	City:
Contact Name :	
Job Title :	
Contact Telephone Number :	Contact E-mail Address :
Quantity Returned / Destroyed :	☐ Credit ☐ Replacement
Signature :	Date :

This form must be returned to BD PAS before this action can be considered closed for your account.



# **CERTIFICATE OF DESTRUCTION**

BD Vacutainer® SST™ II *Advance* Plus Blood Collection Tubes Catalogue number 368879

### PAS-18-1162

If you choose to destroy impacted product, you are requested to complete and return this form to **e-mail** or by fax to: **fax number** as soon as possible or **no later than the 1st February 2018.** 

Dependent on your choice, replacement product or credit processes will be initiated.


Date of Destruction :			
Lot number :			
Quantity:			
Evidence attached :			
Signature :	Date :		

This form, along with evidence, must be returned to BD PAS in order for replacement product or issue of credit processes to be initiated.



## **Appendix 1: Location of Catalogue and Lot Number Information**

## 1. Unit

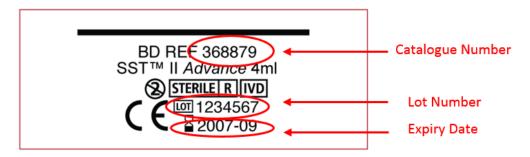

## 2. Shelf Carton



## 3. Shipping Case

