

Drägerwerk AG & Co. KGaA, 23542 Lübeck

To our customers of the  
Dräger Anesthesia Workstations Atlan

April 2026

## Addendum Important Safety Notice!

**Possible failure of automatic ventilation  
due to a fault in the motor electronics**

**ONLY Atlan anesthesia workstations listed in the appendix are affected.  
Other anesthesia devices are NOT affected!**

Dear Sir or Madam

As part of a corrective measure in October 2024, we distributed the **attached important safety notice** and, as a precautionary measure, replaced ventilator motors that had been damaged by a temporary deviation in the assembly process.

In the course of our continuous market monitoring, we have determined that a limited number of additional Atlan devices were affected by this October 2024 notice and, as a precautionary measure, are replacing these ventilator motors.

Please refer to the attached **Important Safety Notice** dated October 2024 and list of affected Atlan devices.

Please ensure that all users of the Dräger Atlan and other affected persons in your company take note of this important safety notice.

For confirmation, please fill out the enclosed reply card and return it within 14 days to us.

If you have made the products available to third parties, please forward a copy of this important safety notice to them.

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23558 Lübeck, Germany  
Postal address:  
23542 Lübeck, Germany  
Tel +49 451 882-0  
Fax +49 451 882-2080  
info@draeger.com  
www.draeger.com

Bank details:  
Commerzbank AG, Lübeck  
IBAN: DE95 2304 0022 0014 6795 00  
Swift code: COBA DE FF 230

Sparkasse zu Lübeck  
IBAN: DE15 2305 0101 0001 0711 17  
Swift code: NOLADE21SPL

Company headquarters: Lübeck  
Commercial register:  
Lübeck Local Court HRB 7903 HL  
General partner:  
Drägerwerk Verwaltungs AG  
Company headquarters: Lübeck  
Commercial register:  
Lübeck Local Court HRB 7395 HL  
UID No. DE135082211

Chairwoman of the Supervisory Board of  
Drägerwerk AG & Co. KGaA: Maria Dietz  
  
Chairman of the Supervisory Board of  
Drägerwerk Verwaltungs AG: Stefan Lauer  
  
Executive Board:  
Stefan Dräger (Chairman)  
Stefanie Hirsch  
Rainer Klug  
Gert-Hartwig Lescow  
Dr. Reiner Piske  
Anton Schrofner

Please keep this important safety notice until the affected components have been replaced. The competent authorities have been informed about this Field Action.

Identification of affected medical devices:

According to our records, you have received at least one Atlan anesthesia workstation manufactured by Drägerwerk AG & Co. KGaA (European Single Registration Number: DE-MF-000005329) that may be affected by this problem. Please refer to the attached list of affected Atlan devices. Our Dräger Service will contact you and arrange an appointment for the replacing the ventilator motor free of charge.

If you have any questions, please do not hesitate to contact your local Dräger representative.

We apologize for any inconvenience this measure may cause you.

Sincerely



Christoph Marquardt  
Product Management  
Therapy Business Unit



Oliver Möller  
Post Market Surveillance  
Quality and Regulatory Affairs

Appendix:

- **Important safety notice**
- **List of affected Atlan devices**
- **Reply card**

### APPENDIX 1: IMPORTANT SAFETY NOTICE from October 2024



Technology for Life

Drägerwerk AG & Co. KGaA, 23542 Lübeck, Germany

To our customers with  
Dräger Atlan workstations

October 2024

### **Important safety notice!**

**Possible failure of the automatic ventilation  
due to an error in the motor electronics**

**This only affects Atlan anaesthesia workstations with serial numbers  
according to the attached list.**

**Other anaesthesia machines are NOT affected!**

Dear Sir or Madam,

In the course of our global market surveillance, we have become aware of cases in which the Atlan anaesthesia workstations either indicated a failure of the piston ventilator before use or suffered a failure of the mechanical ventilation during use. There have not been any reports of patient injuries to date.

Our investigations revealed that the cases are due to an error in the motor electronics, which was caused by a temporary deviation in the assembly process. This may have affected Atlan anaesthesia workstations with the mentioned serial numbers. According to our records, you have received one or more devices from the serial number range affected from us.

**If the error occurs before use, e.g. in standby or system test mode, mechanical ventilation cannot be started. If the error occurs during a procedure, the mechanical ventilation may fail, and the device alerts the user by displaying the "Ventilator error!!!" message. Manual ventilation or spontaneous breathing remains possible in both cases. It may be necessary to manually ventilate the patient to prevent patient injury. With ongoing alarm of the ventilation failure the Atlan continues to provide the Man/Spon ventilation mode, fresh-gas and agent delivery as well as all monitoring functions without restriction.**

Drägerwerk AG & Co. KGaA  
Möslinger Allee 53-55  
23558 Lübeck, Germany  
Postal address:  
23542 Lübeck, Germany  
Tel. +49 451 882-0  
Fax +49 451 882-2080  
info@draeger.com  
www.draeger.com

Bank details:  
Commerzbank AG, Lübeck  
IBAN: DE95 2304 0022 0014 6795 00  
Swift-Code: COBA DE FF 230

Sparkasse zu Lübeck  
IBAN: DE15 2305 0101 0001 0711 17  
Swift-Code: NOLADE21SPL

Registered office: Lübeck  
Commercial register:  
Local court Lübeck HRB 7903 HL  
General partner:  
Drägerwerk Verwaltungs AG  
Registered office: Lübeck  
Commercial register:  
Local court Lübeck HRB 7395 HL  
UID-Nr. DE135082211

Chairman of the Supervisory Board  
for Drägerwerk AG & Co. KGaA  
and Drägerwerk Verwaltungs AG:  
Stefen Leuer

Executive Board:  
Stefan Dräger (chairman)  
Stefanie Hirsch  
Rainer Klug  
Gert-Hartwig Lescow  
Dr. Reiner Piske  
Anton Schrofner



## Technology for Life

Our Dräger Service will contact you and arrange an appointment for replacing the motor, free of charge.

**Until then, please continue operating your devices with the customary care by only using the device under permanent supervision in line with the Atlan instructions for use. If the mechanical ventilation fails, use Man/Spon ventilation mode to ventilate the patient manually as needed.**

If necessary, you can downgrade the alarm priority of the "Ventilator error!!!" alarm with "ALARM RESET".

We regret any inconveniences this may cause.

Please ensure that all users of the Dräger Atlan as well as other persons concerned within your organization are made aware of this Important Safety Notice. Please complete and return the attached reply card to confirm this.

If you have made the products available to third parties, please forward a copy of this Important Safety Notice to them.

Please store this Important Safety Notice at least until the motor has been replaced.

The responsible authorities have been notified of this action.

**Identification of affected medical devices:**

According to our records, you have received at least one Atlan anaesthesia workstation manufactured by Drägerwerk AG & Co. KGaA that may be affected by this issue.

**Contact:**

If you have any questions, please do not hesitate to contact your local Dräger representative. We apologise for any inconvenience this measure may cause.

Yours faithfully,

Christoph Marquardt  
Product Management  
Business Unit Therapy

Oliver Möller  
Post Market Surveillance  
Quality and Regulatory Affairs

**APPENDIX 2:**

Device	Part number	Basic UDI DI
Atlan A100	8621100	040486751301035FK19X030W4
Atlan A100 XL	8621200	
Atlan A300	8621300	040486751301035FK19ZOOOW9
Atlan A300 XL	8621400	
Atlan A350	8621500	040486751301035FK19T010V2
Atlan A350 XL	8621600	

**UDIs for Atlan A100**

(01)4048675594963(11)231025(21)ASSL-0427(93)8621100-04	(01)4048675594963(11)231107(21)ASSM-0033(93)8621100-04
(01)4048675594963(11)231108(21)ASSM-0029(93)8621100-04	(01)4048675594963(11)231107(21)ASSM-0034(93)8621100-04
(01)4048675594963(11)230810(21)ASSJ-0065(93)8621100-03	(01)4048675594963(11)231107(21)ASSM-0035(93)8621100-04
(01)4048675594963(11)231108(21)ASSM-0030(93)8621100-04	(01)4048675594963(11)231107(21)ASSM-0036(93)8621100-04
(01)4048675594963(11)231108(21)ASSM-0031(93)8621100-04	(01)4048675594963(11)231030(21)ASSL-0462(93)8621100-04
(01)4048675594963(11)231108(21)ASSM-0032(93)8621100-04	

**UDIs for Atlan A100 XL**

(01)4048675594956(11)231025(21)ASSL-0422(93)8621200-04	(01)4048675594956(11)231108(21)ASSM-0028(93)8621200-04
(01)4048675594956(11)231108(21)ASSM-0027(93)8621200-04	

**UDIs for Atlan A300**

(01)04048675556343(11)240108(21)ASSN-0304(93)8621300-15	(01)4048675556343(11)230120(21)ASSA-0402(93)8621300-13
(01)04048675556343(11)240108(21)ASSN-0305(93)8621300-15	(01)4048675556343(11)230120(21)ASSA-0403(93)8621300-13
(01)04048675556343(11)240108(21)ASSN-0306(93)8621300-15	(01)4048675556343(11)230120(21)ASSA-0404(93)8621300-13
(01)04048675556343(11)240108(21)ASSN-0307(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0006(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0311(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0007(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0312(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0008(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0313(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0009(93)8621300-13
(01)04048675556343(11)240104(21)ASSN-0237(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0010(93)8621300-13
(01)04048675556343(11)240104(21)ASSN-0238(93)8621300-15	(01)4048675556343(11)230102(21)ASSA-0011(93)8621300-13
(01)04048675556343(11)240104(21)ASSN-0239(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0432(93)8621300-13
(01)4048675556343(11)230711(21)ASSH-0017(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0433(93)8621300-13
(01)4048675556343(11)230720(21)ASSH-0164(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0434(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0314(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0435(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0315(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0436(93)8621300-13
(01)04048675556343(11)240109(21)ASSN-0316(93)8621300-15	(01)4048675556343(11)221222(21)ASRN-0437(93)8621300-13
(01)4048675556343(11)231005(21)ASSL-0012(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0347(93)8621300-13
(01)4048675556343(11)231005(21)ASSL-0013(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0348(93)8621300-13
(01)4048675556343(11)231005(21)ASSL-0014(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0349(93)8621300-13
(01)4048675556343(11)231005(21)ASSL-0015(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0350(93)8621300-13
(01)4048675556343(11)230929(21)ASSK-0349(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0351(93)8621300-13
(01)4048675556343(11)230929(21)ASSK-0350(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0352(93)8621300-13
(01)4048675556343(11)231002(21)ASSK-0351(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0377(93)8621300-13
(01)4048675556343(11)230929(21)ASSK-0352(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0378(93)8621300-13
(01)4048675556343(11)231002(21)ASSK-0353(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0379(93)8621300-13
(01)4048675556343(11)231017(21)ASSL-0225(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0380(93)8621300-13
(01)4048675556343(11)231017(21)ASSL-0226(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0381(93)8621300-13
(01)4048675556343(11)231017(21)ASSL-0227(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0382(93)8621300-13
(01)4048675556343(11)231017(21)ASSL-0228(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0383(93)8621300-13
(01)4048675556343(11)231017(21)ASSL-0229(93)8621300-15	(01)4048675556343(11)230119(21)ASSA-0384(93)8621300-13
(01)4048675556343(11)231006(21)ASSL-0059(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0341(93)8621300-13
(01)4048675556343(11)231006(21)ASSL-0060(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0342(93)8621300-13
(01)4048675556343(11)231006(21)ASSL-0061(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0343(93)8621300-13
(01)4048675556343(11)231006(21)ASSL-0062(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0344(93)8621300-13
(01)4048675556343(11)231006(21)ASSL-0063(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0345(93)8621300-13
(01)4048675556343(11)231009(21)ASSL-0064(93)8621300-15	(01)4048675556343(11)230118(21)ASSA-0346(93)8621300-13
(01)4048675556343(11)231009(21)ASSL-0065(93)8621300-15	(01)4048675556343(11)230105(21)ASSA-0083(93)8621300-13





















































































(01)4048675556183(11)240125(21)ASTA-0275(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0203(93)8621600-13
(01)4048675556183(11)240125(21)ASTA-0276(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0204(93)8621600-13
(01)4048675556183(11)240130(21)ASTA-0352(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0205(93)8621600-13
(01)4048675556183(11)240131(21)ASTA-0354(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0206(93)8621600-13
(01)4048675556183(11)240201(21)ASTA-0364(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0207(93)8621600-13
(01)4048675556183(11)240202(21)ASTA-0395(93)8621600-15	(01)4048675556183(11)230214(21)ASSB-0208(93)8621600-13
(01)4048675556183(11)240202(21)ASTA-0396(93)8621600-15	(01)4048675556183(11)230109(21)ASSA-0128(93)8621600-13
(01)4048675556183(11)240202(21)ASTA-0397(93)8621600-15	(01)4048675556183(11)230414(21)ASSD-0117(93)8621600-15
(01)4048675556183(11)240205(21)ASTA-0407(93)8621600-15	(01)4048675556183(11)230417(21)ASSD-0118(93)8621600-15
(01)4048675556183(11)231204(21)ASSM-0464(93)8621600-15	(01)4048675556183(11)230201(21)ASSB-0001(93)8621600-13
(01)4048675556183(11)231201(21)ASSM-0465(93)8621600-15	(01)4048675556183(11)230201(21)ASSB-0002(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0466(93)8621600-15	(01)4048675556183(11)230103(21)ASSA-0031(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0467(93)8621600-15	(01)4048675556183(11)230104(21)ASSA-0059(93)8621600-13
(01)4048675556183(11)231201(21)ASSM-0468(93)8621600-15	(01)4048675556183(11)230104(21)ASSA-0060(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0469(93)8621600-15	(01)4048675556183(11)230418(21)ASSD-0149(93)8621600-15
(01)4048675556183(11)231204(21)ASSM-0491(93)8621600-15	(01)4048675556183(11)221219(21)ASRN-0346(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0492(93)8621600-15	(01)4048675556183(11)221219(21)ASRN-0347(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0493(93)8621600-15	(01)4048675556183(11)221219(21)ASRN-0345(93)8621600-13
(01)4048675556183(11)231204(21)ASSM-0494(93)8621600-15	(01)4048675556183(11)230106(21)ASSA-0099(93)8621600-13

### Ventilator kit assembly

Manufacturing month	Serial numbers	
01-2023	ASSA-...	...-0396, ...-0465
02-2023	ASSB-...	...-0224, ...-00233, 0768,0769
03-2023	ASSC-...	...-0653, ...-0666, ...-0670, ...-0711, ...-0721
04-2023	ASSD-...	...-0333, ...-0334
05-2023	ASSE-...	...-0003, ...-0008, ...-0009, ...-0011, ...-0041, ...-0057, ...-0059, ...-0060, ...-0064, ...-0099
06-2023	ASSH-...	...-0258, ...-0259, ...-0264-0268, ...-0270-0274, ...-0276, ...-0279, ...-0280

## Reply card

Please fax or email this form to your Dräger representative!

<b>D R Ä G E R</b>	To: < To be completed by relevant subsidiaries/dealers >
	Department: _____
	Dräger Representative: _____
	Fax: _____
	Phone: _____
	E-mail: _____

Re: **Supplement Important safety information Dräger Atlan, February 2026**

(Please complete)

<b>C U S T O M E R  D A T A</b>	Hospital: _____
	Customer name: _____
	Phone: _____ Fax: _____
	Email: _____
	Address: _____
	Address 2: _____
	Country: _____
	Town/City: _____
<b>Quantity of affected units :</b> _____	

We acknowledge receipt of the safety notice and that the information contained therein has been brought to the attention of all affected users.

(Please complete and sign)

Title/Position: _____
Name: _____ (Please print in capitals )
Signature: _____ Date: _____